Octopus

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Dear Customer,

Thank you for choosing our product and for the trust you have placed in us. In order for you and the user to enjoy your new product safely, conveniently, and comfortably, we kindly ask that you read the Instructions for Use. Doing this will also help avoid damage, injury, and incorrect operation and handling of the product. Please ensure that a copy of the Instructions for Use remains with the product so that every user has access to it.

If you have any questions, please contact your authorised distributor or us directly.

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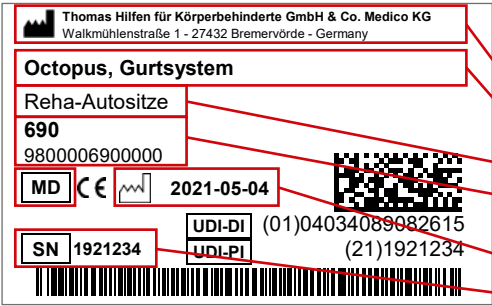
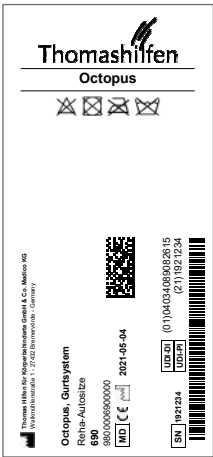
We reserve the right to make technical changes. This can lead to differences between your product and how it is presented in brochures and other media.

Table of Contents

Figures	2
Preface / Manufacturer	14
Labels / Model Identification	15
General Remarks / Safety Instructions	16
Intended Purpose	17
– Intended Use / Application Areas	17
– Indication / Patient Target Group / Intended Users	17
– Risks Associated with Use and Contraindications	18
Product Description	18
– Scope of Delivery / Standard Equipment	18
– Putting into Service	18
– Teaching the User	19
Adjustment Options / Features	19
– Accessories	19
– Technical Data	20
Cleaning, Maintenance, and Repair	20
– Cleaning, Care Instructions, and Disinfection on a Daily Basis	20

– Maintenance and inspection by the distributor / Maintenance Schedule . . .	21
– Spare Parts	22
– Storage / Disposal	22
EU Declaration of Conformity	22
Warranty	22
Period of Use / Service Life	23
Reuse	23
Distributor	23

Labels / Model Identification

Label	Meaning
 <p>The image shows a detailed barcode label with several fields highlighted by red boxes and lines pointing to a legend. The fields include: <ul style="list-style-type: none"> Manufacturer: Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG, Walkmühlenstraße 1 - 27432 Bremervörde - Germany Product Name: Octopus, Gurtsystem Product Group: Reha-Autositze Item Number short / long: 690 and 9800006900000 MD (Medical Device) symbol and CE mark Manufacture Date: 2021-05-04 UDI-DI (01)04034089082615 UDI-PI (21)1921234 Serial Number (SN): 1921234 </p>	<p>Barcode Label (Detail)</p> <p>Position on the Product (Fig.)</p> <ul style="list-style-type: none"> - Manufacturer - Product Name - Product Group - Item Number short / long - MD = Medical Device - Manufacture Date - Serial Number
 <p>The image shows a textile label with a barcode label attached. The textile label includes: <ul style="list-style-type: none"> Logo: Thomashilfen Product Name: Octopus Three crossed-out symbols (X) Manufacturer: Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG, Walkmühlenstraße 1 - 27432 Bremervörde - Germany Product Name: Octopus, Gurtsystem Product Group: Reha-Autositze Item Number: 690 and 9800006900000 MD symbol and CE mark Manufacture Date: 2021-05-04 UDI-DI: (01)04034089082615 UDI-PI: (21)1921234 Serial Number (SN): 1921234 </p>	<p>Textile Label with Barcode Label</p> <p>Position on the Product (Fig. 1)</p>

Remarks

General Remarks

- Read the Instructions for Use carefully before first use. If you have difficulty reading, please have them read to you. If the copy of the Instructions for Use is missing, you can download one from www.thomashilfen.de. The contents of the digital Instructions for Use can be enlarged.
- Check the packaging for any transport damage and to be sure the product is in flawless condition.
- Changes to the product are only permitted within the scope of the intended adjustment options.
- Cleaning and daily care can be provided by relatives and caregivers. Maintenance and inspection are done by the distributor.
- All labels have to remain on the product and should not be tampered with.

Safety Instructions

- Check at least every 3 months that the settings on the product still work for the user's current body measurements. If need be, have a distributor / orthopaedic technician adjust the settings.
- Do regular (weekly) checks to make sure that the setting options on the product work.
- Please do not overload the product and note the maximum user weight and the maximum load capacity (see technical data).
- Up to a height of 150 cm / 59.1" or up to the age of 12, the positioning harness must be used in conjunction with a booster seat.
- Never leave the user unattended with the product.
- The product should only be used by one user at a time. No other individuals should be placed in or on the product.
- The positioning harness can be used in conjunction with a booster seat without backrest. Make sure that the harness is always attached to the vehicle seat. Therefore, only place the booster seat on the car seat after the positioning harness has been mounted!
- All users have to be familiar with how the product works. This way in an emergency the user can be removed quickly from the product.
- Make sure that the supplied safety hammer with built-in belt cutter is within reach of the rider and/or caretaker!

- Do not lock the belt buckles on the waistcoat buckles unless absolutely necessary.
- ALWAYS secure the user with the 3-point safety harness located in the vehicle. The harness used is NOT safety relevant and is only used for positioning.
- Intense sunlight and heat can lead to high surface temperatures on the product. As a result, there is a risk of injury when touching the product. Only use the product once it has cooled down enough. In general, try to protect the product from direct sunlight.
- Please note the safety standards which apply to your vehicle as outlined in the vehicle's Instructions for Use to ensure the safety of the user.
- If there is a serious and unexpected accident with the product, please inform our company immediately and the responsible authorities.

Intended Purpose

Intended Use / Application Areas

The Octopus rehab positioning harness is a class 1 medical device.

It is designed to safely transport the physically disabled and to ensure the passenger is sitting securely in vehicles. The passenger is always fastened using the vehicle's own three-point seat belt. The seven-point seat waistcoat in the rehab positioning harness helps to position and support the passenger user in an upright sitting position.

The rehab positioning harness provides additional support to help the passengers sit securely. It can be combined with a standard booster seat that is appropriate for the passenger's weight. The clasps of the positioning harness can be locked if desired to prevent unwanted opening.

Any use that goes above and beyond this does not comply with the intended use.

Indication / Patient Target Group / Intended Users

The rehab positioning harness is for children, adolescents, and adults with mild structural and / or functional limitations or poor head and torso posture. In addition, it is suitable for passengers with psychomotor agitation and / or sensory disorders. It provides a stable, physiological position if a standard seat belt does not sufficiently allow the passenger to sit upright in the vehicle.

Risks Associated with Use and Contraindications

If used properly, risks can be eliminated during use. There are no known contraindications.

Product Description

Scope of Delivery

Your product is delivered with the standard equipment and comes preassembled. Any accessories ordered along with the product are included and have to be assembled subsequently. A copy of the Instructions for Use is enclosed with the product.

Standard Equipment

Base back part; 7-point seat vest with belt buckles; 2x shoulder straps; 4x pelvis belts; crotch strap; 7x lockable side release buckles; keys for belt buckles; emergency hammer with seatbelt cutter; water-repellent cushions

Putting into Service

- Carefully remove the transport protection and the packaging.
- Keep the packaging material and small parts out of the reach of children to prevent swallowing or choking.

Attaching the base back part

Place the base back part, with the Thomashilfen logo readable, on the backrest of the car seat. Put the headrest of the car seat in a high position for easier installation of the harness. Then thread the two buckles with the inscription "Press" through the bars of the headrest. Then adjust the headrest again.

Next, the lower metal buckles and the crotch belt are threaded between the back and seat of the car seat. If necessary, folding the backrest down will make assembly easier.

Finally, the metal buckles on the back are clicked in. Then the base back part is secured by tightening the strap underneath the buckles. The crotch strap is threaded through the hinged buckle.

Caution: If your car seat does not have a gap between the backrest and the seat area, you will need the "belt extension" accessory to use the product safely.

Fig. 2: Threading the head section

Fig. 3: Threading the foot section

Fig. 4: Clicking in metal buckles

Fig. 5: Tightening the metal buckles

Teaching the User

The user is taught how to use this device by trained staff who will reference these Instructions for Use.

Trained staff should install the product in the vehicle with the user and adjust the seat to the measurements of the actual user.

The user has to be shown all of product's functions as well as how to install and remove it from the vehicle.

Adjustment Options / Features

Attaching and adjusting the 7-point seat vest

The 7-point seat vest is connected to the basic back part.

The straps are threaded through the buckles on the back of the basic back part. Then the chest size of the vest is reduced or increased by these straps..

Fig. 6: Threading a strap

Fig. 7: Tightening the strap

Warning: The standard three-point safety belt must always be used for the safe and legally permitted transport of persons in the car!

Accessories

Belt extension

The belt extension consists of a belt for the horizontal fixation of the system (one buckle) and a belt for the extension of the buckles (two buckles).

The lower buckles of the base back are adjusted to the maximum length and placed on the seat. The two buckles are now connected to the corresponding counterpart of the belt extension. Then the buckles, with the crotch strap, are pulled under the complete car seat and clicked into place on the back with the buckles of the headrest.

Now the small belt (one buckle) is placed in front of the base back and the crotch belt. The belt is connected to the back of the car seat. The belt is then tightened as deeply as possible on the backrest.

Finally, the basic back is tightened and the crotch strap is adjusted. (Fig. 5)

Note: Make sure that the straps do not cross or twist.

Fig. 8: Connecting the belt extension

Fig. 9: Guide of the belts under the seat

Fig. 10: Mounting the side belt

Fig. 11: Guide of the crotch belt

Technical Data

Octopus positioning harness

Cushion 7-point seat vest (W x H)	34 x 42 cm / 13.4 x 16.5"
Cushion base back part (W x H)	34 x 84 cm / 13.4 x 33.1"
Body height user	130 - 185 cm / 51.2 - 72.8"
User weight	30 - 80 kg / 66.1 - 176.4 lbs


Cleaning, Maintenance, and Repair

Cleaning, Care Instructions, and Disinfection on a Daily Basis

Padding

The water-repellent padding can be cleaned with a sponge and warm water. Then dry them with a towel.

Please also note the following care instructions.

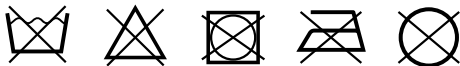Do not use a washing machine! This can damage the product and the washing machine.

We would like to point out that even high-quality materials can fade as a result of long and intense exposure to sunlight or frequent washing.

Plastic Parts

The plastic parts on the product can be cleaned and disinfected with standard soap (neutral soap).

Metal Parts

The metal parts on the product can be cleaned and disinfected with standard soap (neutral soap).

Disinfection

Clean the padding, plastic parts, and metal parts before disinfecting.

Damp wipe all the plastic and metal parts on the product with disinfectant. Only water-based products should be used. When disinfecting note the directions from the manufacturer in question.

The padding has to be disinfected before reusing. To do so use the right detergent in keeping with the guidelines of the Association of Applied Hygiene VAH (Verbund für Angewandte Hygiene e.V.).

Maintenance and inspection by the distributor

Preparation

A copy of the Instructions for Use has to be enclosed with the product. If one is not available, request a copy from the distributor. Familiarise yourself with how the product works. If you are not familiar with the product, read the Instructions for Use before testing it. Clean the product before starting to test it. Please note the care instructions in the Instructions for Use and product-specific test instructions. If you have further questions, please contact the staff member you dealt with or Thomashilfen directly.

Service and repairs to the product should only be done by qualified staff. It is the user's responsibility to ensure that the prescribed inspections / maintenance are done regularly and in a timely fashion. If damage is apparent, the user has to actively inform and have the damage fixed immediately by qualified staff.

Maintenance Schedule – Annual Inspection

	Area	Function	Damage	Replace
1	Adjustment options	👁 / ↻	👁 / ↻	
2	Padding	👁 / ↻	👁 / ↻	
3	Belts	👁 / ↻	👁 / ↻	
4	Plastic parts	👁 / ↻	👁 / ↻	
5	Metal parts	👁 / ↻	👁 / ↻	
6	Emergency hammer with belt cutter	👁 / ↻	👁 / ↻	
7	Labels		👁 / ↻	
done by:			on:	
👁 = Inspection ↻ = Reuse ✓ = Test completed				

Spare Parts

Only original spare parts should be used for repairs. A list of the spare parts is available at www.thomashilfen.de. To receive the right spare parts or for possible technical recalls, we normally require the serial number / UDI-PI and the exact UDI-DI.

Removal / installation of spare parts should only be done by qualified staff! In the event of returns, please make sure the product is securely packaged for transportation.

Storage

Store the product in a safe place when it is not being used. Do not lay or put any heavy objects on the product while it is being stored. Protect the product from weather like rain, intense sunlight, and frost. If the product is wet, wipe the metal and plastic parts dry and let the padding dry before storing the product.

Disposal

If the product is no longer usable due to its condition or it has reached the end of its service life, it can be recycled at the local waste disposal company. Please refer to the local guidelines for recycling and disposal.

EU Declaration of Conformity

We hereby declare as the party solely responsible that this product and any versions and accessories comply with the basic regulations. The current EU Declaration of Conformity can be downloaded at www.thomashilfen.de or requested from us directly.

Warranty

Thomashilfen offers you the two-year warranty as required by law for this product. The warranty period starts when the item is handed over. The warranty includes all claims which impact function. Not included is damage which is the result of improper use (e.g. overloading) and natural wear and tear.

Period of Use

The product has a period of use of up to 6 years if used correctly and serviced regularly by a professional as per the information from the manufacturer. The product can be used beyond this point if it is in safe and technically sound condition.

The theoretical period of use does not equate to a guaranteed service life. It depends on the intensity of use and always undergoes an individual assessment by the distributor. The period of use ends before the given time frame if the user has reached the maximum body weight or maximum body height to safely use the product.

The period of use does not apply to wear parts like padding, covers, etc.

Service Life

The product has a service life of 8 years or 2 reuses (3 users) if the requirements stipulated for the period of use are met. The service life consists of the period of use and the storage times between reuses.

Reuse

The product is suitable to be reused. Before being passed on to the next user the product always has to be carefully cleaned and disinfected. The condition has to be checked by trained staff for any wear and damage. Any issues have to be remedied to ensure the continued safe use of the product. A copy of the Instructions for Use is a component of the product and has to be passed on to the new user.

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Tendercare Ltd.